

## Directions For Use (Continued)

### 7. Disinfection, cleaning and sterilization

Not applicable.

### 8. Shelf Life

Shelf life of product, 5 years from date of manufacture.

Manufactured for:

**DENTSPLY**  
**MAILLEFER**

DENTSPLY International, Inc.  
608 Rolling Hills Dr.  
Johnson City, TN 37604  
Phone: 1-800-924-7393  
Fax: 1-800-924-7389  
www.dentsply.maillefer.com  
Made in Switzerland

MAX-I-PROBE® is a registered  
trademarks of DENTSPLY  
International Inc.

F19 02151.US / 00 / 2014 / created 10/2014

## Directions For Use

**DENTSPLY**  
**MAILLEFER**

**Max-i-Probe®**

Periodontal/Endodontic irrigating probes

Rx Only  
For Dental Use Only

STERILE R

STERILIZED BY  
RADIATION



FOR SINGLE  
USE ONLY



CAUTION:  
SEE DIRECTIONS  
FOR USE



OPENED  
PACKAGES  
ARE NOT REPLACED



DO NOT USE  
IF SEAL  
BROKEN

STERILE EO

STERILIZED USING  
ETHYLENE OXIDE

### 1. Indications for use

MAX-I-PROBE® is indicated to irrigate periodontal pockets and root canals.

### 2. Contraindications

None known.

## Directions For Use (Continued)

### 3. Warnings

Sterile unless tamper proof seal is broken and individual cartridge is opened or damaged. Examine individual cartridge pack for integrity of seal prior to use. Do not use if tamper proof seal is broken or cartridge is opened or damaged. **Do not bend probe more than 25° during use. SINGLE USE ONLY.** If used more than once there is a cross contamination risk to the user and patient. **DO NOT STERILIZE BEFORE USE.** The product is sterile, if resterilized, the probe may separate from the hub during use.

### 4. Precautions

To prevent needle sticks, keep probe covered with protector until ready to use. Retain sheath to remove needle after use and dispose in approved sharps container. U.S. federal law restricts this device to sale to or by order of a dentist.


### 5. Adverse Reactions

None known.

## Directions For Use (Continued)

### 6. Step-by-step Instructions

- 1) Grasp needle cartridge and twist. Tamper evident seal will break.

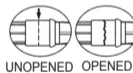

**IMPORTANT!**  
**DO NOT BEND OR SNAP OPEN**

- 2) After twisting, pull cap straight off sheath cartridge. Keep probe covered with protector until ready for use.



- 3) TO ATTACH NEEDLE:

Grasp by needle sheath and attach to luer tip. Use a 3ml syringe. **Turn until completely secure.** Make sure tip is securely fastened before use.



- 4) Remove sheath and retain to remove needle after use.
- 5) After use dispose in approved sharps container.